

# First aid for burns.



\* Excludes product and packaging.

## aura<sup>3</sup>: Burns kits with BurnSoothe™ Dressings

*Aura3 represents a first for the first aid industry, the first 100% recycled and fully recyclable first aid kit box on the market.*

*The fresh new box design feature colour coded, embossed, and screen-printed lids. The new graphics stand out from a distance, enabling quick identification in an emergency, especially in a busy workplace. Compact and convenient, the integral bracket allows easy placement and fixing.*

### **BURNSOOTHE COOLS, MOISTENS, AND SOOTHES**

*Sterile trauma hydrogel burn dressings, designed to minimise trauma and skin damage. Non-adherent to burnt skin.*

*The individual kits provide fast and effective treatment for burns, scalds and sunburn in the workplace or home.*

*A fully stocked kit designed to provide fast and effective treatment for burns in the workplace or home. BurnSoothe™ dressings provide a fast, effective treatment for burns, while minimising trauma and skin damage. These kits are packed with a range of different sizes to suit different possible burn injuries.*



**Integral bracket**



**Single clip**



**Embossed cross**



**Easy identification**



**Tamper seal**



**Laser etching**

**Better for the workplace, better for the planet.**




+44 (0) 08456 448808 | sales@reliancemedical.co.uk | www.reliancemedical.co.uk

**RELIANCE MEDICAL**  
*'Leading First Aid'*




REV150223


# Types of burn.

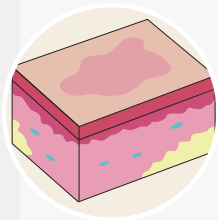


**SUPERFICIAL:**  
Red and dry, with no blisters.

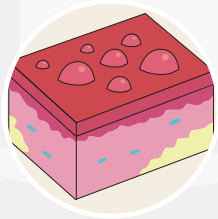

**PARTIAL THICKNESS:**  
Red, blistered or swollen.

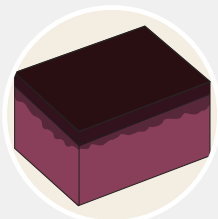

**FULL THICKNESS:**  
Charred, leaking or white skin.

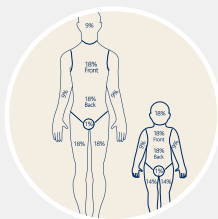

Call an ambulance for any partial or full thickness burns, chemical burns, suspected internal electrical burns or any deep burns larger than 5% of the body area.

## How to apply a BurnSoothe™ dressing.

The individual kits provide fast and effective treatment for burns, scalds and sunburn in the workplace or home.

### HEAT BURN

- Open BurnSoothe™ sachet and lay the dressing across the wound site.
- Squeeze the remaining gel from the sachet across the dressing.
- Secure BurnSoothe™ in place with a conforming bandage, taking care not to apply too much pressure. Seek medical attention if necessary.

### CHEMICAL BURN

- Wash the wound site continually in cold water for approx. 15-20 mins.
- Follow the instructions as per heat burn shown above.



## BURNSOOTHE FIRST AID KITS



2030 BURNS FIRST AID KIT in XS Orange Aura3 Box  
 SIZE: 20.5cmH x 20cmW x 6.5cmD  
 UNIT: 1  
 MOQ: 20



124 BURNS FIRST AID KIT in Small Orange Aura3 Box  
 SIZE: 21cmH x 23.5cmW x 10cmD  
 UNIT: 1  
 MOQ: 10

CONTENTS	124	2030
Bandage, Conforming 5cm x 4m	–	1
Bandage, Conforming 7.5cm x 4m	1	1
Bandage, Conforming 10cm x 4m	1	–
Burn Dressings, 2.5cm x 5cm	–	4
Burn Dressings, 10cm x 10cm	2	3
Burn Dressing, 5cm x 15cm	1	–
Burn Dressing, 20cm x 20cm	1	–
Burn Gel, Sachets 3.5g	8	8
Burn Gel, 125ml Bottle	1	–
Eye Wash, Pods 20ml	–	2
Foil Blanket	–	1
Gloves, (Pair)	1	1
Guidance Leaflet	1	1
Shears	1	1
Tape, Microporous	–	1



When you see this symbol with our products, you are guaranteed we have reduced plastic in the mould, used 100% recycled materials, 100% recyclable product.



+44 (0) 08456 448808 | sales@reliancemedical.co.uk | www.reliancemedical.co.uk

**RELIANCE MEDICAL**  
 'Leading First Aid'

