



Integral bracket



Single clip



Embossed icon



Laser etching, no labels



Includes eye bath



and eye pad dressings



the right Eye Wash kit.

Our New Aura3 eyewash stations are better for the workplace and better for the planet, providing quick and easy access to Reliwash isotonic eyewash solution in the event of eye injury. Aura3 represents a first for the first aid industry, the first 100% recycled and fully recyclable first aid kit box on the market.

EYE-CATCHING

The fresh new box design feature colour coded, embossed, and screen-printed lids. The new graphics stand out from a distance, enabling quick identification in an emergency, especially in a busy workplace. Compact and convenient, the integral bracket allows easy placement and fixing.

EYE-OPENING

Eye injuries can occur in any work environment. Our Eyewash stations are available in double and triple sizes for all workplace requirements.

Most eye or skin incidents are caused by dust, dirt, metal and wood splinters, and chemicals such as oil and solvents. In these circumstances, rinsing the eye or skin with a solution of 0.9 % sterile sodium chloride will be sufficient. The isotonic solution is suitable for rinsing and cleansing the eyes or skin as a means of first aid in case of injury.

Better for the workplace, better for the planet.

Easy to use.







Place the casualty in a comfortable position with their head inclined, so the affected eye is lower than the other eye.



Pour the fluid from the inner corner of the eye outward for 10 seconds at a time, allowing the casualty to blink in between until the object is removed.



If irrigating the eye does not work, cover the injury with a sterile eye pad dressing and seek medical attention immediately. Covering the eye will limit eye movement.

Contaminant:

DIRT, DUST, METAL, WOOD, SPLINTERS

- **Action**: To rinse the eyes and prevent foreign substances coming into contact with them and causing damage.
- **Directions**: Rinse until the substance is removed, or medical help arrives.

Contaminant:

OIL AND SOLVENTS

- Action: To rinse the eyes until the unpleasant sensation is over.
- **Directions**: Continue rinsing until you receive medical attention.



EYE WASH STATIONS





904	DOUBLE EYE WASH STATION in Medium Blue Aura3 Box
SIZE:	25.5cmH x 28cmW x 10cmD
UNIT:	1
MOO:	10





903	TRIPLE EYE WASH STATION in Medium Blue Aura3 Box
SIZE:	25.5cmH x 28cmW x 10cmD
UNIT:	1
MOO	10

CONTENTS	904	903
	DOUBLE	TRIPLE
Dressings, Eye Pad	2	3
Eye Wash, Bottles 500ml	2	3
Eye Wash, Pods 20ml	2	_
Mirror	1	_



RELIANCE MEDICAL *'Leading First Aid'*